

*Resilient like they
should be...*



*The FlowBoot Protects, Supports &
Promotes Full Movement & Circulation*



FlowBoot
True Support and Circulation

Introducing the FlowBoot

Reduce Injuries & Improve Performance

The FlowBoot is a combination of a **protective open-cell foam boot** paired with a **reinforcing waterproof wrap**.

Together, the boot and wrap provide your horse with support and protection in the areas most prone to injuries. **Designed around anatomical equine structure**, the FlowBoot ensures natural blood circulation and a full range of motion and flexibility.

Boot = Heat-absorbing foam liner with protective outer foam

Wrap = .75mm neoprene lined lycra elastic wrap 4 3/8" x 84"

FlowBoot Benefits:

- 1 Opening in front creates bridge to relieve pressure from the digital extensor tendon
- 2 Extra cushion and support to dissipate direct pressure on the superficial digital flexor tendon
- 3 Supports and protects the ligaments of the proximal sesamoid bones to reduce load and hyperextension
- 4 Grooves correspond to and create bridges over the palmar digital veins ensuring proper blood circulation
- 5 45 degree angle allows for proper application of wrap


Approximately 80% of all equine injuries occur in the area that the FlowBoot protects



Introducing the FlowBoot

Instructions

The FlowBoot is
available in
sets of 2 or 4
9 fun colors
XS | S | M | L | XL

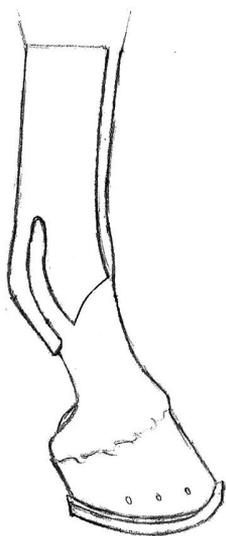

Step 1:


Apply foam boot
to the horse's lower leg

Step 3:

Wear the FlowBoot for
added protection during
competitions, races and
leisure activity.


Step 4:

Wash the FlowBoot as
needed and reuse.

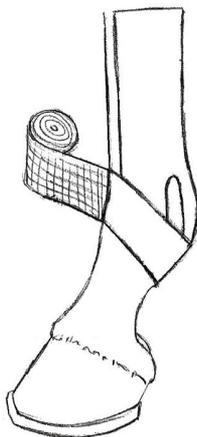

Step 2:

Apply the wrap as you would a
polo wrap. Starting in the middle
of the cannon bone, securely
apply the wrap around the boot,
wrapping down to the fetlock joint
with a 45 degree layer that cups
the ankle and wrapping back up
the leg to secure the boot with
additional support. Fasten wrap
with velcro strip.

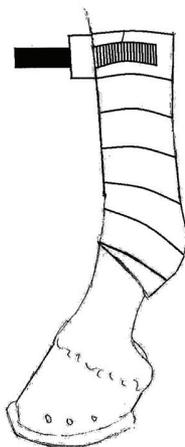

Caution: Do not apply the wraps directly to the horse or injury may occur. The wraps of the FlowBoot should only be used when paired with the protective foam boots.

www.flowboot.com